

**WATER FOR IRRIGATION**  
**Distilled water sterile apyrogenic**  
**External use only**

**Composition**

Sterile and apyrogenic water  
PE vials are DEHP free

**Intended use**

Sterile water for irrigation

Sterile water exerts a mechanical cleansing action for sterile irrigation of body cavities, tissues or wounds and for washing, rinsing or soaking surgical dressings, instruments and laboratory specimens.

**Contra-indications**

Not for injections.

Do not use for irrigation that may result in absorption into the blood. It is hypotonic and will cause hemolysis

**Warnings and precautions for use**

Before use, verify container integrity and that the solution is clear.

Do not use if there is leaking or if the solution is cloudy or contains particles.

It's not recommended for use during surgical procedures.

**Method of administration**

EXTERNAL USE, only for irrigation. DO NOT INJECT.

**Instruction for use**

Twist cap smoothly in direction of the arrow to break the seal and remove the cap.  
Disposable container.

**Precautions for storage**

Store at a temperature not exceeding +25°C. Do not freeze.

Do not use after the date printed on the label.

**Shelf-life**

4 years

**Reference numbers**

**4807372:** 250 ml in transparent PE bottle/box of 10

**4807389:** 500 ml in transparent PE bottle/box of 10

**4807395:** 1000 ml in transparent PE bottle/box of 10

**Distributor**

Laboratoires CHAIX ET DU MARAIS (CDM Lavoisier), 7 rue Labie, 75017 Paris, France

Tel: +33 1 55 37 83 83 Fax: +33 1 55 37 83 84

**Leaflet date :** July 2010